

# MANUAL

**BIX-BIX-H70-2 Senior male catheterization model**



**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**



## BIX-BIX-H70-2 Senior male catheterization model

### A. Features

The product is based on the basis of clinical nursing and nursing practice technical outline of the requirements by the company's newly developed Advanced male catheterization model, this model is made of plastic material, made by die cast molding, injection, puncture, genitals By imports of soft plastic. Has a vivid, true operation, easy accessibility, reasonable structure, durability and other characteristics

### B. Product Features

1. Male catheterization
2. Male bladder lavage
3. Ostomy drainage
4. Intramuscular injection
5. Femoral intramuscular

### C. Men catheterization

Before catheterization operation, first instillation of water 200 ml, will lift the penis to simulate membranous urethra bent portion straight, and reached through the catheter, there is a real sense of obstruction. When the bladder is inserted approximately 20 cm, the "urine" from the catheter discharge about 200 ml.

### D. Scope

1. Medical University, School of Nursing, School of Health, clinical teaching in medical colleges teach students practice and training.
2. Medical University, School of Nursing, School of Health, clinical teaching in medical colleges teach students practice and training.
3. Universal basic health units Clinical Training

## **E.Maintenance Warranty**

1. After enema, catheterization operation training and other functions, to empty residue when operating.
2. When a long time, the model should be wiped clean, packaged and placed large cool dry place for long life

**Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd.**

**Ada Med Supply Limited**

---

Shanghai Chinon Medical Model & Equipment Manufacturing Co.,Ltd. Building 3,  
No.1288 kungang Road, Xiaokunshan Town, Songjiang District, Shanghai, China

Tel: +86-13383897707

Fax: 021—57722933

Post code: 201612

E-mail: [edith@adahealthy.com](mailto:edith@adahealthy.com)

Website: [www.adahappy.com](http://www.adahappy.com)